

Padded Belts 4130, 4135 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS: FOR CHAIR USE:			
1. Position the patient as far back in the seat as possible, with the buttocks against the back of the chair.			
2. Lay the lap belt across the patient's thighs with the foam facing in.			
3. Bring the ends of the connecting straps down at a 45-degree angle between the seat and the wheelchair sides.			
4. Criss-cross the straps behind the chair and draw them around the opposite side kick spurs.			
5. Kneel next to the back wheel, outside (not directly behind) the wheelchair, as it is too difficult to push the slide buckles if positioned directly behind the wheelchair.			
6. Adjust the tightness of the slide buckles. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the chair is adjusted.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
7. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			
PRODUCT REMOVAL: 1. Stand behind the chair and push the slide buckles close to the kick spurs to create excess slack.			
2. Pull the lap belt over the patient's head or down over the patient's knees.			