

Breezeline Pelvic Holder 4411 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS:			
1. Lay the pelvic holder on the chair, with the narrow side to the back of the chair. The label will be face down in the seat. To assist with application, bunch the wider end together.			
2. Bring the ends of the connecting straps on the narrow end, down between the seat and the wheelchair sides.			
3. Criss-cross the straps and use quick-release ties to attach the straps to the opposite side kick spurs, out of the patient's reach.			
4. Position the patient as far back in the seat as possible, with the buttocks against the back of the chair.			
5. Bring the wide part of the pelvic holder up between the patient's legs.			
6. Bring the ends of the connecting straps on the wide end, down between the seat and wheelchair sides at a 45-degree angle.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
7. Criss-cross the straps and use quick-release ties to attach straps to the opposite side kick spurs, out of the patient's reach.			
8. If the chair has an adjustable seat, secure straps to a movable part of the chair frame, out of the patient's reach.			
9. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the chair is tilted or adjusted.			