

Pelvic Holders 4427, 4430 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS:			
1. Lay the pelvic holder on the chair with the narrow side to the back of the chair. The label will be face down in the seat.			
2. Bring the ends of the connecting straps on the narrow end, down between the seat and the wheelchair sides			
3. Secure the straps behind the backrest.			
4. Position the patient as far back in the seat as possible, with the buttocks against the back of the chair.			
5. Bring the wide part of the pelvic holder up between the patient's legs.			
6. Bring the ends of the connecting straps on the wide end, down between the seat and wheelchair sides at a 45-degree angle.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
7. Secure the straps behind the backrest as low as possible.			
8. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the chair is tilted or adjusted.			