

Self-Releasing Wrap-Around 3658 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS: FOR WHEELCHAIR, GERI-CHAIR OR RECLINER-CHAIR USE:			
1. Position the patient on the belt as far back in the seat as possible, with the buttocks against the back of the chair.			
2. Wrap the foam belt around the patient and chair backrest. Press the hook and loop fastener firmly together.			
3. Secure the hook and loop fastener in front of the patient for self-release, or behind the patient for assisted-release. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			
4. Place the belt over the lower lap to hold the buttocks against the backrest and to help hinder sliding.			
5. Wrap the belt under the seat on a part of the frame that will not allow the belt to slide forward.			
FOR HOSPITAL BED USE:			
1. Wrap the belt underneath and around a movable part of the bed frame.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
2. Lay the belt over the top of the patient, and press the hook and loop fastener firmly over the patient to secure product. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			
3. For self-release, place the fastener in reach of the patient. For assisted-release, place the fastener out of the reach of the patient. Check that the belt is secure and will not change position, loosen, or tighten if the patient pulls, or if the bed is adjusted.			