

# Foam Heel Guard 6127 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
<b>APPLICATION INSTRUCTIONS:</b> 1. Unfasten hook and loop straps.			
2. Orient the Heel Guard with the long side positioned under the leg.			
3. Place the patient's heel in the Heel Guard so that the heel is situated over the heel cutout.			
4. Fasten in place with the hook straps. Check to make sure circulation is not impaired by making sure straps are not too tight on the leg.			
5. Ensure that Heel Guard cannot be easily removed from the patient's leg.			
6. Follow your Hospital's policy and procedures for assessing patient while the product is in use.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at [www.posey.com](http://www.posey.com).

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • [www.posey.com](http://www.posey.com)