

Locking Twice-As-Tough™ Cuffs 2792, 2793, 2798, 2799 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
2792 Wrist, pair, key lock (color-coded blue) 2793 Ankle, pair, key lock (color-coded red) 2798 Wrist, pair, key lock cuff and strap (color-coded blue) 2799 Ankle, pair, key lock cuff and strap (color-coded red)			
APPLICATION INSTRUCTIONS: ⚠ WARNING Connecting Straps (repeat steps 1-3 for each side) Before each use, check cuffs and straps for cracks, tears, and/or excessive wear or stretch; cracked or broken buckles or locks; and/or that hook and loop adheres securely, as these may allow patient to remove cuff. Discard if device is damaged. If you have any questions about patient safety, ask the appropriate medical authority for alternatives.			
1. Position the patient on the bed.			
2. Bring the ends of the strap down through the inside of the side rails so they do not interfere when the side rails are raised. Thread the end of the strap over the top, around the frame, and through the lock. Make sure that the straps are attached to a movable part of the bed frame, out of the patient's reach.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
3. Pull the strap snug and close the lock. Check that the lock “clicks” shut. If a lock is not completely closed, it can pop open. Before leaving the patient’s side, test the lock by trying to open it without the key.			
Applying the Cuffs (repeat steps 1-3 for each limb)			
1. Wrap the neoprene piece (the blue or red side should be positioned against the skin) around the wrist/ankle. Attach the black hook and loop pieces together, followed by the blue or red hook and loop pieces. The fuzzy piece should be sandwiched between the two pieces of hook. Be sure to overlap at least one inch (3 cm).			
2. Press the hook and loop closure together firmly and make sure it adheres securely. Slide ONE finger (flat) between the cuff and the inside of the patient’s wrist to ensure proper fit. The cuffs must be snug enough to prevent escape, but not interfere with circulation.			
3. 2792, 2793 - Pass the end of the limb strap over the top of the cuff and through the two D-rings on the cuff. Bring the strap back over the first ring and through the two D-rings on the cuff. 2798, 2799 - Pass the end of the short strap through the lock and adjust it to the desired length. Check that the lock “clicks” shut. If a lock is not completely closed, it can pop open. Before leaving the patient’s side, test the lock by trying to open it without a key.			
To Unlock Cuffs:			
1. Insert the Posey Key into the lock and turn counterclockwise. This will prevent jamming.			
⚠WARNING			
Posey Keys MUST be readily available to staff at all times. In an emergency, it is difficult to cut the cuffs with scissors to free the patient.			
NOTE: Universal Posey Keys fit all Posey locks. To order extra keys, contact Posey at 1.800.447.6739, and ask for Cat. #1074.			