

Twice-As-Tough™ Hobble 2801 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS:			
⚠ WARNING			
Before each use, check cuffs and straps for cracks, tears, and/or excessive wear or stretch; cracked or broken buckles or locks; and/or that hook and loop adheres securely, as these may allow patient to remove cuff. Discard if device is damaged.			
If you have any questions about patient safety, ask the appropriate medical authority for alternatives.			
Follow these steps to apply device (repeat steps 1-7 on the other ankle):			
1. Position the cuff behind the patient's ankle with the opening to the front. The connecting strap with label "FRONT" should be visible from the front.			
2. Wrap the neoprene piece around the ankle. Attach the black hook and loop pieces together followed by the red hook and loop pieces. The fuzzy loop piece should be sandwiched between the two pieces of hook. Be sure to overlap the piece at least one inch (3 cm).			
3. Press the hook and loop closure together firmly and make sure it adheres securely.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
4. Pass the end of the cuff strap over the top of the cuff and through the lock.			
5. Pull the strap snug and close the buckle.			
6. Check for proper fit by inserting one finger between the cuffs and the patient's ankles. The cuffs must be snug enough to prevent escape, but not interfere with circulation.			
7. Check that the lock "clicks" shut. If a lock is not completely closed, it can pop open. Before leaving the patient's side, test the lock by trying to open it without the key.			
To Unlock Cuffs: 1. Insert the Posey Key into the lock and turn counterclockwise. This will prevent jamming. ▲WARNING Posey Keys MUST be readily available to staff at all times. In an emergency, it is difficult to cut the cuffs with scissors to free the patient.			
NOTE: Universal Posey Keys fit all Posey locks. To order extra keys, contact Posey at 1.800.447.6739, and ask for Cat. #1074.			