

# Connected Locking Cuffs 2212, 2213 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
<b>2212</b> Connected Wrist Set - Synthetic Leather <b>2213</b> Connected Ankle Set – Synthetic Leather			
<b>APPLICATION INSTRUCTIONS:</b> <b>⚠WARNING</b> Before each use, check cuffs and straps for cracks, tears, and/or excessive wear or stretch; cracked or broken buckles or locks; and/or that hook and loop adheres securely, as these may allow patient to remove cuff. Discard if device is damaged.  If you have any questions about patient safety, ask the appropriate medical authority for alternatives.			
Follow these steps to apply device 1. Attach the bed strap on each side, to a movable part of the bed frame, at a comfortable level for either wrist or ankle use.			
2. Lay the patient down on the bed on top of the connecting strap between the cuffs.			
3. Wrap the cuff around the patient's limb.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at [www.posey.com](http://www.posey.com).

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • [www.posey.com](http://www.posey.com)



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
4. Insert the metal post through the most suitable adjustment hole.			
5. Secure the lock to the post.			
6. Check for proper fit by inserting one finger between the cuffs and the patient's wrist or ankles. The cuffs must be snug enough to prevent escape, but not interfere with circulation.			
7. Check that the lock "clicks" shut. If a lock is not completely closed, it can pop open. Before leaving the patient's side, test the lock by trying to open it without the key.			
<p><b>To Unlock Cuffs:</b></p> <p>1. Insert the Posey Key into the lock and turn counterclockwise. This will prevent jamming.</p> <p><b>▲WARNING</b></p> <p>Posey Keys MUST be readily available to staff at all times. In an emergency, it is difficult to cut the cuffs with scissors to free the patient.</p>			
<p><b>NOTE:</b> Universal Posey Keys fit all Posey locks. To order extra keys, contact Posey at 1.800.447.6739, and ask for Cat. #1074.</p>			