

# Connected Cuffs 2204, 2205, 2206, 2207 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
<b>2204</b> Connected Synthetic Leather Wrist Cuffs <b>2205</b> Connected Synthetic Leather Ankle Cuffs <b>2206</b> Connected Leather Wrist Cuffs <b>2207</b> Connected Leather Ankle Cuffs			
<b>APPLICATION INSTRUCTIONS:</b> <b>⚠WARNING</b> Before each use, check cuffs and straps for cracks, tears, and/or excessive wear or stretch; cracked or broken buckles or locks; and/or that hook and loop adheres securely, as these may allow patient to remove cuff. Discard if device is damaged.  If you have any questions about patient safety, ask the appropriate medical authority for alternatives.  Make sure the patient is unable to reach the bed connecting straps and free themselves.			
Follow these steps to apply device (repeat steps 1-5 on other limb): 1. Attach the bed strap on each side, to a movable part of the bed frame, at a comfortable level for either wrist or ankle use.			
2. Lay the patient down on the bed on top of the connecting strap between the cuffs.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at [www.posey.com](http://www.posey.com).

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • [www.posey.com](http://www.posey.com)



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
3. Wrap the cuff around the patient's limb.			
4. Bring the U-bar through the closest vertical slot and then pass the tongue through the U-bar.			
5. Check for proper fit by inserting one finger between the cuffs and the patient's wrist/ankle. The cuffs must be snug enough to prevent escape, but not interfere with circulation.			