

Clean Cuffs 2214, 2215, 2216 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
2214 Synthetic Leather Locking Wrist 2215 Synthetic Leather Locking Ankle 2216 Posey Crisis Bag with one set of 2214 and 2215			
APPLICATION INSTRUCTIONS: ⚠WARNING Before each use, check cuffs and straps for cracks, tears, and/or excessive wear or stretch; cracked or broken buckles or locks; and/or that hook and loop adheres securely, as these may allow patient to remove cuff. Discard if device is damaged. If you have any questions about patient safety, ask the appropriate medical authority for alternatives.			
NOTE: Keep excess strap out of reach to prevent patient from pulling strap and damaging or opening lock.			
Follow these steps to apply device (repeat steps 1-7 on other limb): 1. Wrap the cuff around the patient's limb.			
2. Insert the metal post through the most suitable adjustment hole.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
3. Secure the lock to the post.			
4. Check for proper fit by inserting one finger between the cuffs and the patient's wrist or ankles. The cuffs must be snug enough to prevent escape, but not interfere with circulation.			
5. Check that the lock "clicks" shut. If a lock is not completely closed, it can pop open. Before leaving the patient's side, test the lock by trying to open it without the key.			
6. Wrap the strap around the juncture of a movable part of the bed frame, out of the patient's reach. Ensure that the strap is wrapped around the frame at least once before passing the end of the strap through the lock. Leave no more than 1" (3 cm) of excess strap sticking out of the lock. Pulling on excess strap may allow the patient to damage the lock.			
7. Pull firmly on the strap to verify that the lock is secure.			
<p>To Unlock Cuffs:</p> <p>1. Insert the Posey Key into the lock and turn counterclockwise. This will prevent jamming.</p> <p>⚠WARNING</p> <p>Posey Keys MUST be readily available to staff at all times. In an emergency, it is difficult to cut the cuffs with scissors to free the patient.</p>			
<p>NOTE: Universal Posey Keys fit all Posey locks. To order extra keys, contact Posey at 1.800.447.6739, and ask for Cat. #1074.</p>			