



Mitt Checklist 2817

Facility Information

Facility Name:

Street Address:

City / State
/ Zip Code:

Instructor:

Signature:

Contact Information

Name:

Title:

Nursing Unit:

Date:

Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently

Description	Competency Level	Date	Comments
1. The Posey 2817 is a padded mitt with mesh and finger slots. Orient the mesh side up.			
2. Insert the patient's hand into the mitt, palm down. Separate the fingers so that three fingers are positioned in the middle slot and the smaller finger and thumb have their own slot.			
3. Wrap the wrist strap around the smallest part of the patient's wrist, through the plastic ring, and secure it with the hook and loop fastener.			
4. The strap should be snug, but not compromise circulation. Slide one finger (flat) between the device and the inside of the patient's wrist to ensure proper fit.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com