

Mitts 2811, 2911 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS:			
1. The Posey 2811 or 2911 is a padded mitt with an inspection flap on one side. Orient the inspection flap side up.			
2. Insert the patient's hand into the mitt, palm down.			
3. For the 2811: Wrap the wrist strap around the smallest part of the patient's wrist, through the fabric loop and then through the plastic ring. Secure it with the hook and loop fastener. For the 2911: Wrap the wrist strap around the smallest part of the patient's wrist, through fabric loop and secure it with the hook and loop fastener.			
4. For the 2811: The strap should be snug, but not compromise circulation. Slide one finger (flat) between the device and the inside of the patient's wrist to ensure proper fit. For the 2911: Ensure proper fit. The strap must be snug, but not compromise circulation.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
5. To view the hand/fingers: a. Reach under the inspection flap, detach the hook and loop fastener, and pull back on the flap to expose the back of the hand.			
b. To close, tuck the inspection flap into the end of the mitt and press the hook and loop closure together firmly.			