



Mitt Checklist 2813

Facility Information

Facility Name:

Street Address:

City / State
/ Zip Code:

Instructor:

Signature:

Contact Information

Name:

Title:

Nursing Unit:

Date:

Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently

Description	Competency Level	Date	Comments
1. The Posey 2813 is a padded mitt with mesh on one side. Orient the mesh side up.			
2. Insert the patient's hand into the mitt, palm down. Separate the fingers so each finger fits into its own slot.			
3. Close the padded cuff around the smallest part of the patient's wrist and secure it with the hook and loop fastener.			
4. Pull the mesh flap of the mitt down over the wrist cuff and secure the quick-release buckle. Insert two fingers (flat) under the buckle and adjust the buckle strap so the mitt is snug, but does not compromise circulation.			
5. Slide one finger (flat) between the cuff and the inside of the patient's wrist to ensure proper fit.			
6. The following steps should be done when using the optional bed connecting straps (hospital bed use only) to help prevent the patient from removing the device or inflicting self-injury:			

Description	Competency Level	Date	Comments
A. Secure the strap using a Quick-Release Tie. Wrap the strap around the outer side of the movable part of the bed frame, out of the patient's reach.			
B. Bring the strap across and then behind the vertical strap to form a loop. Take the end of the strap, form a second loop and bring it through the first loop.			
C. Pull down to tighten and tuck any excess strap out of reach.			
D. To release the straps, pull on the tail of the strap.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com