



Posey®

Mitt Checklist 2815

Facility Information

Facility Name:

Street Address:

City / State
/ Zip Code:

Instructor:

Signature:

Contact Information

Name:

Title:

Nursing Unit:

Date:

Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently

Description	Competency Level	Date	Comments
1. The Posey 2815 is a padded mitt with open finger slots. Orient the finger slot side up.			
2. Insert the patient's hand into the mitt, palm down. Separate the fingers so each finger fits into its own slot.			
3. Wrap the wrist strap around the smallest part of the patient's wrist, through the plastic ring, and secure it with the hook and loop fastener.			
4. The strap should be snug, but not compromise circulation. Slide one finger (flat) between the device and the inside of the patient's wrist to ensure proper fit.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • Web: www.posey.com