



Mitt Checklist 2820

Facility Information

Facility Name:

Street Address:

City / State
/ Zip Code:

Instructor:

Signature:

Contact Information

Name:

Title:

Nursing Unit:

Date:

Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently

Description	Competency Level	Date	Comments
1. The Posey 2820 has a rigid plate on one side. Orient the rigid side down.			
2. Insert the patient's hand into the mitt, palm down.			
3. The wrist strap should be positioned around the smallest part of the patient's wrist. Pull the strap around the patient's wrist.			
4. The strap should be snug, but not compromise circulation. Slide one finger (flat) between the device and the inside of the patient's wrist to ensure proper fit.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com