

Roll Jacket 3320 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS: 1. Place the device on the patient with the opening in the back. Make sure side seams are located under the arms. If they are not, the sizing may be incorrect.			
2. Close with the zipper.			
3. Secure straps to the wheelchair or bed frame, out of the patient's reach.			
4. Slide an open hand (flat) between the device and the patient to ensure proper fit. The device must be snug, but not interfere with breathing.			
FOR WHEELCHAIR USE: 1. Position the patient as far back in the seat as possible with the buttocks against the back of the chair.			
2. Bring the straps over the hips at a 45-degree angle and pass down between the seat and the wheelchair sides.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
3. Criss-cross the straps, and use quick-release ties to attach straps to the opposite side kick spurs, out of the patient's reach.			
4. If the chair has an adjustable seat, secure straps to a movable part of the chair frame, out of the patient's reach.			
FOR BED USE:			
1. Bring one of the straps through the square opening on the same side, threading it under the zipper and out of the jacket through the square opening on the opposite side. Be sure that the strap lies UNDER the zipper, not over.			
2. Repeat Step 1 (FOR BED USE) with the other strap.			
3. Secure straps with quick-release ties to a movable part of the bed frame at waist level, out of the patient's reach.			