

# Houdini Security Suit 3420 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
<b>APPLICATION INSTRUCTIONS:</b>			
1. Place the device on over the patient's head with the pelvic piece hanging down the back.			
2. Bring the pelvic piece up between the patient's legs and pull toward the front.			
3. Take the ends of the connecting straps and thread them through the vertical slot in the middle of the pelvic piece.			
4. Separate the straps, taking one to the right and the other to the left, and thread through the loops that connect the top vest portion and the pelvic piece.			
5. Secure straps to the wheelchair or bed frame, out of the patient's reach.			
6. Slide an open hand (flat) between the device and the patient to ensure proper fit. The device must be snug, but not interfere with breathing.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at [www.posey.com](http://www.posey.com).

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • [www.posey.com](http://www.posey.com)



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
<b>FOR WHEELCHAIR USE:</b> 1. Position the patient as far back in the seat as possible with the buttocks against the back of the chair.			
2. Bring the straps over the hips at a 45-degree angle and pass down between the seat and the wheelchair sides.			
3. Cross the straps, and use quick-release ties to attach straps to the opposite side kick spurs, out of the patient's reach.			
4. If the chair has an adjustable seat, secure straps to a movable part of the chair frame, out of the patient's reach.			
<b>FOR BED USE:</b> 1. Secure the straps, using quick-release ties, to a movable part of the bed frame at waist level, out of the patient's reach.			